

BACTRACK® B70 Quick Start Guide

Be sure to read the entire owner's manual and follow procedures carefully before using this device.

STEP 1



WAIT 20 MINUTES

**NO
EATING**



**NO
DRINKING**



**NO
SMOKING**



Wait at least 20 minutes after eating, drinking or smoking before blowing into the tester.

This is the recommended procedure for all breath alcohol testers.

STEP 2

CORRECT



NOT CORRECT



Take a deep breath and exhale with **mouth WIDE OPEN as close as possible to the BACTRACK** without actually touching your lips to it. Exhale for 5 FULL SECONDS.

STEP 3

DO NOT DRINK AND DRIVE



The BACTRACK® Breathalyzer helps estimate Blood Alcohol Content (BAC). A single drink can impair coordination and judgement and users should never drink and drive.

This device is intended to measure alcohol in the human breath. Measurements obtained by this device are used as an aid in the detection of alcohol intoxication. For additional information please consult the included owner's manual.

BACTRACK® SERVICE GUIDE

[illegible]